

FastTrack™ Liner Use Guide



horsepower[™]
TECHNOLOGIES

Congratulations on your purchase of Horsepower Technologies FastTrack™. Please take a moment to read these instructions before using FastTrack™.

You can find step-by-step videos online at <http://www.horsepowertech.com/support>

A pair of Horsepower Technologies approved Liners have been included with this order of FastTrack™. To purchase additional Liners please contact your distributor or Customer Support.


For questions or comments please contact:

Customer Support
844-51-HORSE
support@horsepowertech.com
or visit www.horsepowertech.com


WARNING:

1. Correct and accurate fit and application is critical to this device's proper performance.
 - Carefully read instructions and warnings prior to use.
2. FastTrack™ is designed to be used during elective exercise activities from stall rest through moderate canter. Consult your veterinarian regarding safe, appropriate activity levels and duration while your horse wears FastTrack™.
3. Do not apply Liner or FastTrack™ if your horse has open wounds or sores on their cannon or pastern.
4. Do not apply any topical ointments or insecticides prior to applying Liner or FastTrack™.
5. Immediately discontinue use and consult your veterinarian if your horse experiences pain, swelling, skin irritation or any adverse reactions while using Liner or FastTrack™.
6. Horses are prone to sudden movements, please exercise caution while applying and fitting Liner and FastTrack™.
7. Do not use Liners if they are damaged (torn, ripped). Discontinue use if Liners are showing significant wear and tear.


Liner Features

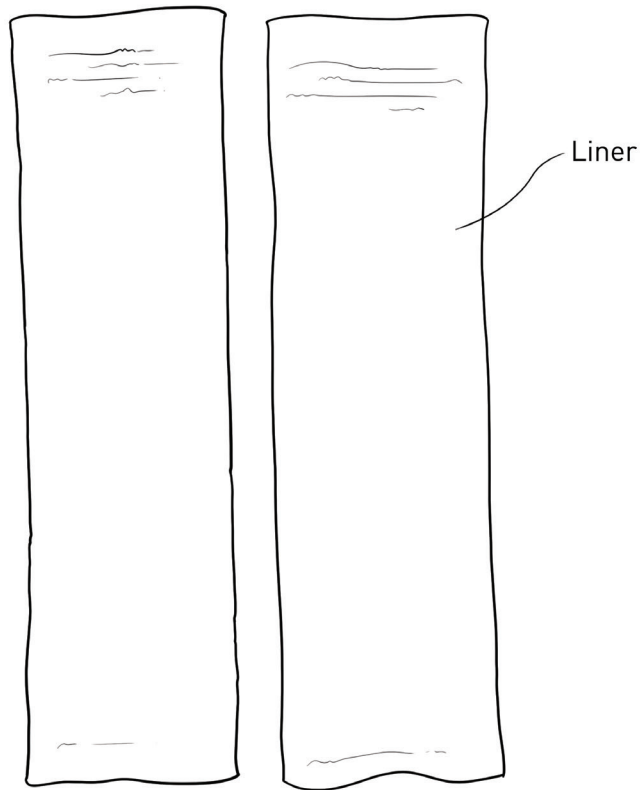

Forelimb Anatomy

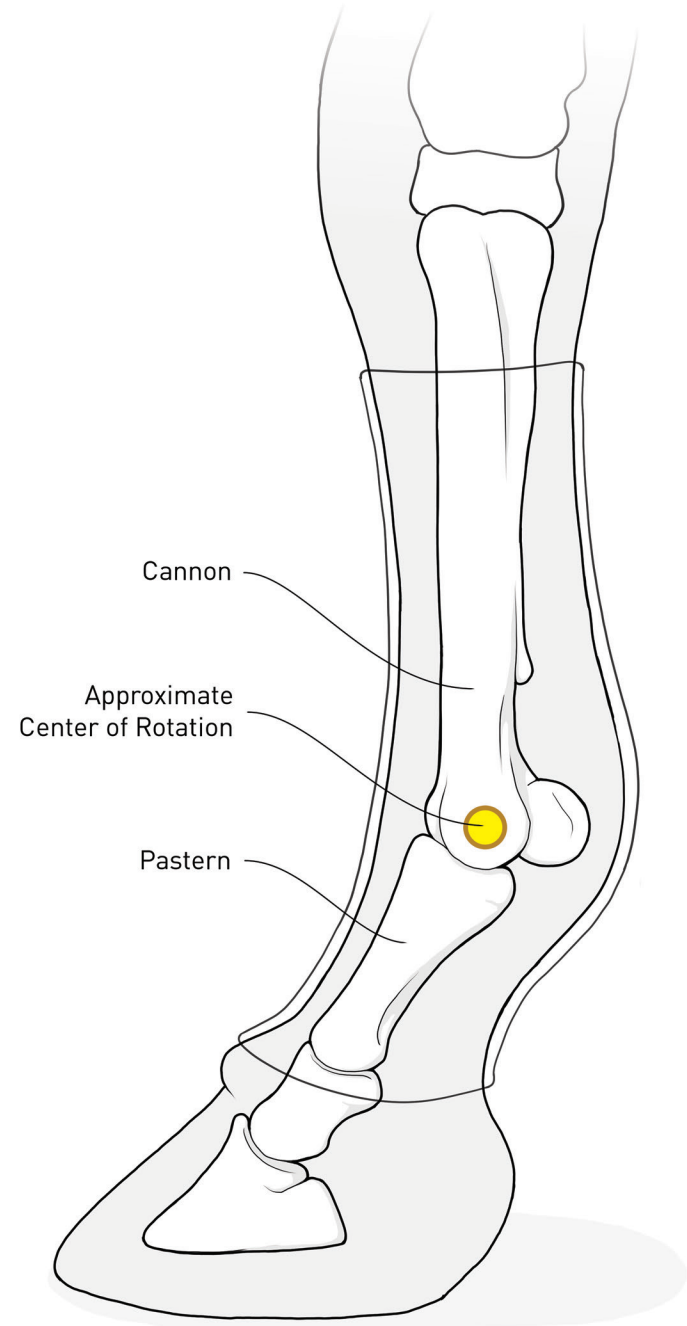


1 Prepare Leg for Liner Application

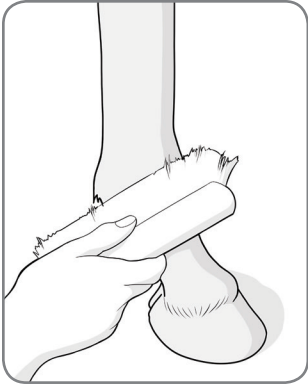


1. Have the horse stand square on a firm, flat surface.
2. Brush the horse's leg from the knee down using a hard bristle brush.
3. Inspect the horse's legs to make sure they are clean, dry and free of abrasions or irritations.

2 Apply Liner to Forelimb

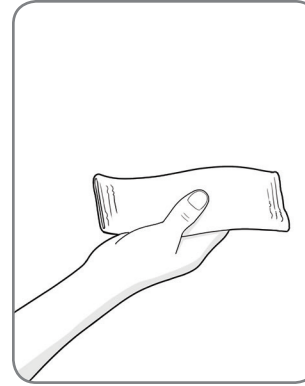

1. Select a Liner from the storage bag. The Liners are not left / right leg specific.

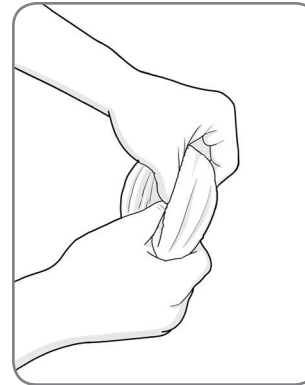

2. Open and roll up the Liner.



3. Pick up the leg to apply the Liner.

2 Apply Liner to Forelimb cont.




4. Pull the Liner completely over the hoof.




5. Once the Liner is over the horse's hoof set the leg down.


3 Place Liner in Desired Location

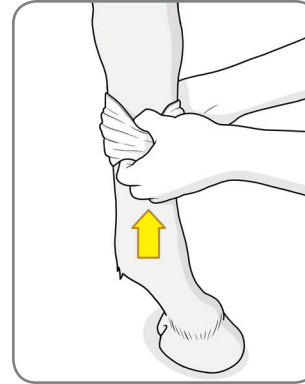

1. Grip the Liner with both hands and gently pull it up over the fetlock and cannon until it reaches the base of the knee.

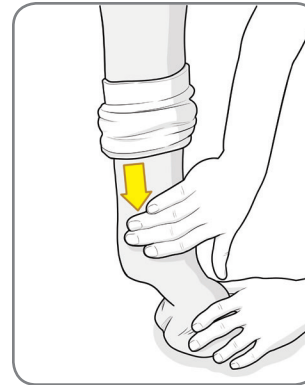

2. Using both hands, gently smooth out any hair that was displaced during Liner repositioning.

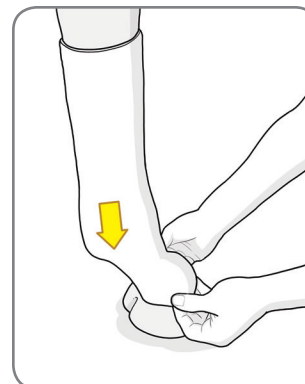


3. Unroll the Liner and pull the bottom edge downwards so it covers the full length of the cannon, fetlock and pastern.

Note: The top edge of the Liner should touch the bottom of the knee and the bottom edge of the Liner should rest just over the hoof when properly applied depending on the length of the forelimb.


Note: Take care to ensure the hair is not displaced during Liner application.

3 Place Liner in Desired Location cont.

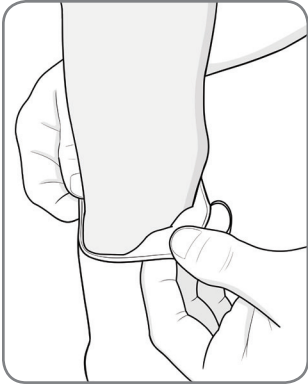


4. Once the Liner has been applied to the leg, go back and adjust the Liner so it is touching right below the horse's knee.

Note: Pull the Liner away from the leg and up to avoid roughing the hair.

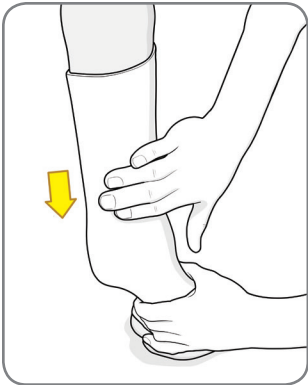


5. Using your hands, smooth out wrinkles or bumps in the Liner using a downwards motion.

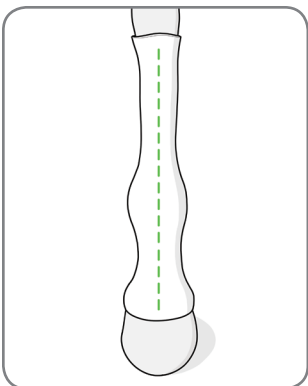


6. Ensure that the Liner is straight top to bottom and not twisted on the forelimb.
7. Repeat all steps on the opposite forelimb.


4 Apply FastTrack™ to Forelimb

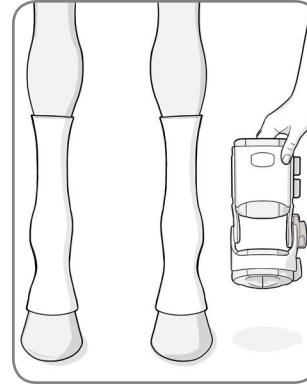

1. Referring to the FastTrack™ Daily Use Guide, apply FastTrack™ to the desired forelimb over Liner.

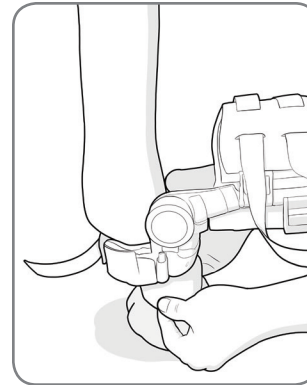


2. When applying lower cuff to pastern, take care to ensure that the bottom of the Liner does not wrinkle or travel up on the pastern. If the bottom of the Liner has traveled up, pull it back down so it covers the pastern.

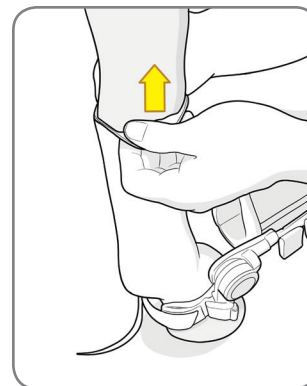


3. Check the top of the Liner to make sure it did not slip on the cannon. If the top of the Liner did slip during lower cuff application, pull it back up so it touches the bottom of the horse's knee.

4 Apply FastTrack™ to Forelimb cont.

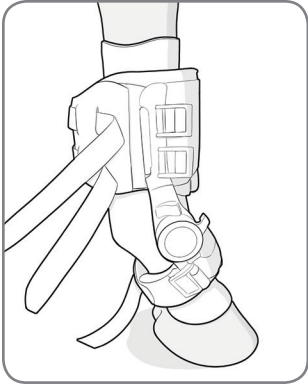


4. Apply the upper cuff to the cannon ensuring that the Liner does not fall or bunch during application.

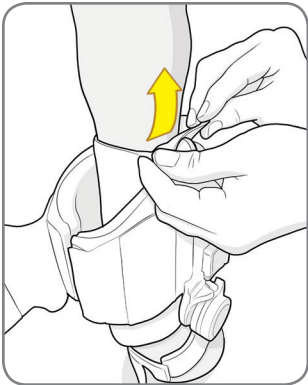


5. It may be necessary to pull the top edge of the Liner up after applying the upper cuff to smooth out any bunching that may have occurred during application.
6. Repeat all steps on the opposite forelimb.

5 Remove FastTrack™ and Liner

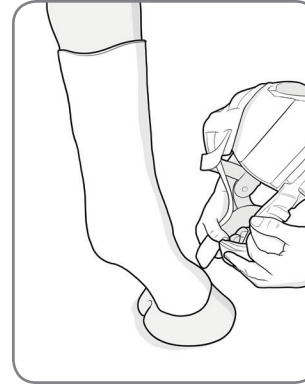

1. Referring to the FastTrack™ Daily Use Guide, remove FastTrack™ from the horse's forelimbs.

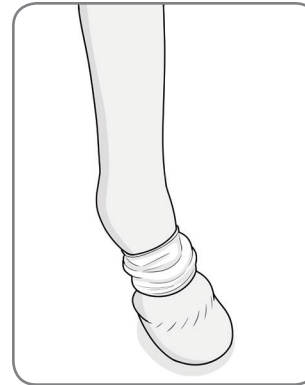

2. Pull the top edge of the Liner down on the forelimb so the Liner is gathered around the pastern.



3. Using two hands, pick up the leg, grip the Liner and pull it over the horse's hoof for removal.
4. Repeat all steps on the opposite forelimb.

Storage and Care Instructions

After Every Use:	<ul style="list-style-type: none">• Inspect the Liner for wear or perforation. If a Liner shows significant signs of wear or perforation, discard pair and use a new pair of Liners for next exercise session.• Remove dirt and debris using a soft bristle brush
Store In:	<ul style="list-style-type: none">• Store in a cool dry area in tote bag provided with FastTrack™• Label the name of the horse on the bag as each pair of Liners may be specific to individual horses
Cleaning:	<ul style="list-style-type: none">• Clean and dry as needed• Wipe with damp sponge and air dry

To order Liners, please contact your distributor or Horsepower Customer Support.

For Product Warranty information, please refer to our website: www.horsepowertech.com

Customer Support
844-51-HORSE
support@horsepowertech.com
or visit www.horsepowertech.com



For questions and support,
please contact customer support:

844-51-HORSE

support@horsepowertech.com

visit www.horsepowertech.com

Horsepower Technologies
110 Canal Street, Suite 311
Lowell, MA 01852 USA

Version 1.0
526-03-057